

BD Molecular Swab Collection Kit: Vaginal swab specimen self-collection procedure






For clinician staff

- Do not use a lubricant with the BD Molecular Swab Collection Kit to aid in self-collection.
- Patient must collect their specimen before any vaginal exam is performed with a lubricant.
- Self-collected vaginal swabs are approved specimen types for BD CTGCTV2 for BD MAX™ System and BD MAX™ Vaginal Panel

Patient instructions for self-collection

Please read all instructions before collecting specimens. If you have any questions about this procedure, please ask your doctor or nurse.

- Wash hands with soap and water. Rinse and dry.
1. Remove the sterile swab from its sheath, taking care not to contaminate the tip or shaft. Carefully pull the cap with attached swab off the tube. Do not touch the soft tip or lay the swab down. If you touch or drop the swab tip or the swab is laid down, discard the swab and request a new vaginal swab. Check for presence of the swab tip. If the swab has no tip, discard it and request a new vaginal swab.
 2. Hold the swab by the cap with one hand so the swab tip is pointing toward you (Figure 2). With your other hand, gently spread the skin outside the vagina. Insert the tip of the swab into the vaginal opening (Figure 2). Point the tip toward your lower back and relax your muscles.
 3. Gently slide the swab no more than 2 inches (5 cm) into the vagina (Figure 3). If the swab does not slide easily, gently rotate the swab as you push. If it is still difficult, do not attempt to continue self-collection; consult your clinician at this point.
 4. Rotate the swab for 10 to 15 seconds (Figure 4).
 5. Withdraw the swab without touching the skin outside the vagina. Place the swab in the sheath and cap the sheath securely (Figure 5).
- After collection, wash hands with soap and water, rinse, and dry. Return the swab in its sheath to the nurse or clinician as instructed.

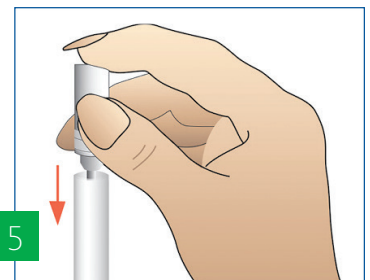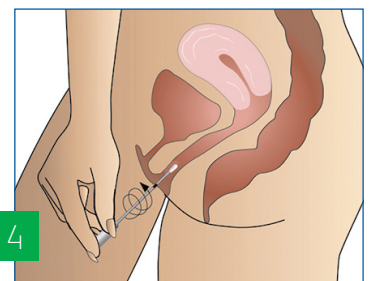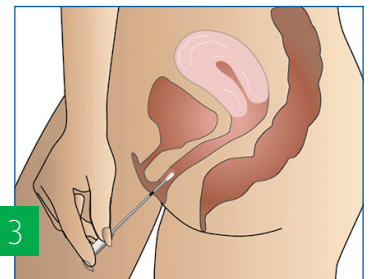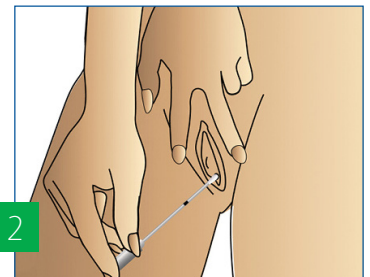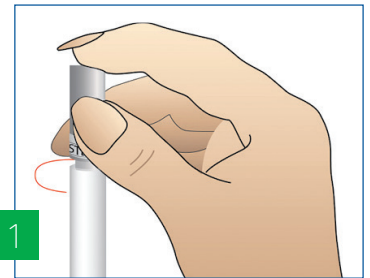

BD - Europe, Terre-Bonne Park - A4, Route de Crassier 17, 1262 Eysins, Switzerland

The BD MAX™ System, BD Vaginal Panel and BD Molecular Swab Collection Kit are in vitro diagnostic medical device bearing a CE-mark.

The BD CTGCTV2 & BD CTGC2 for BD MAX™ System are in vitro diagnostic medical devices bearing a CE mark and is CE certified by BSI Group The Netherlands B.V. (Notified Body Number = 2797)

bd.com



BD, the BD Logo and MAX are trademarks of Becton, Dickinson and Company or its affiliates.
© 2021 BD. All rights reserved. BD-39666

